

## How to use the Intercept 2 mouth swab


Do not have anything to eat, drink or smoke  
10 minutes prior to testing

- 1 Remove mouth swab from pouch
- 2 Keep swab under tongue until indicator turns blue. Ensure indicator (see yellow arrow) is facing upwards. Average collection time is 2-3 minutes



Indicator will turn BLUE when enough sample has been collected

- 3 Place swab into specimen tube and close screw top lid. Fill in Laboratory form and **place security seal over specimen tube.** Place specimen tube **RED LID facing down** into transport tube

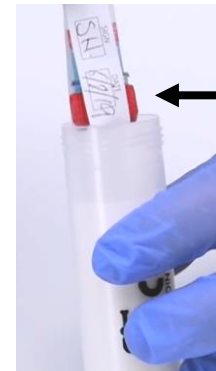

Red lid on specimen tube **MUST** face down to fit correctly into transport tube

